



OCS-Plus USER GUIDE

More information on: www.ocs-test.org/ocs-plus


Using OCS-Plus

Register device by clicking 'Register' and scanning your licensed QR code



Click to add text

A pop up will indicate license details for **you** to confirm

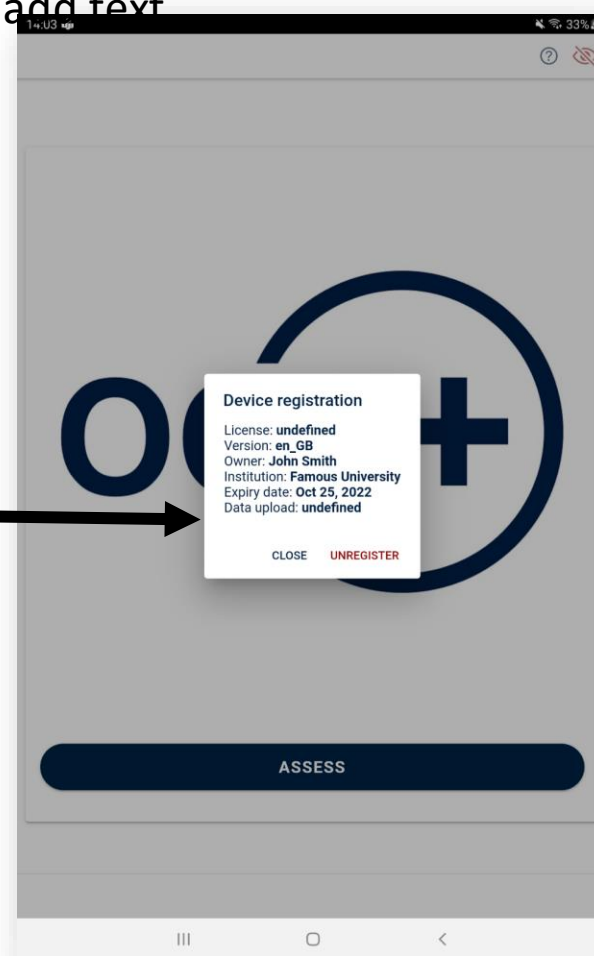

You can now start an assessment



Using OCS-Plus


⏩ Skip a subtest


⏸ pause an assessment

You read aloud these instructions

ID of the respondent


 Respondents should see NEXT screen

 Respondents should **not** see NEXT screen

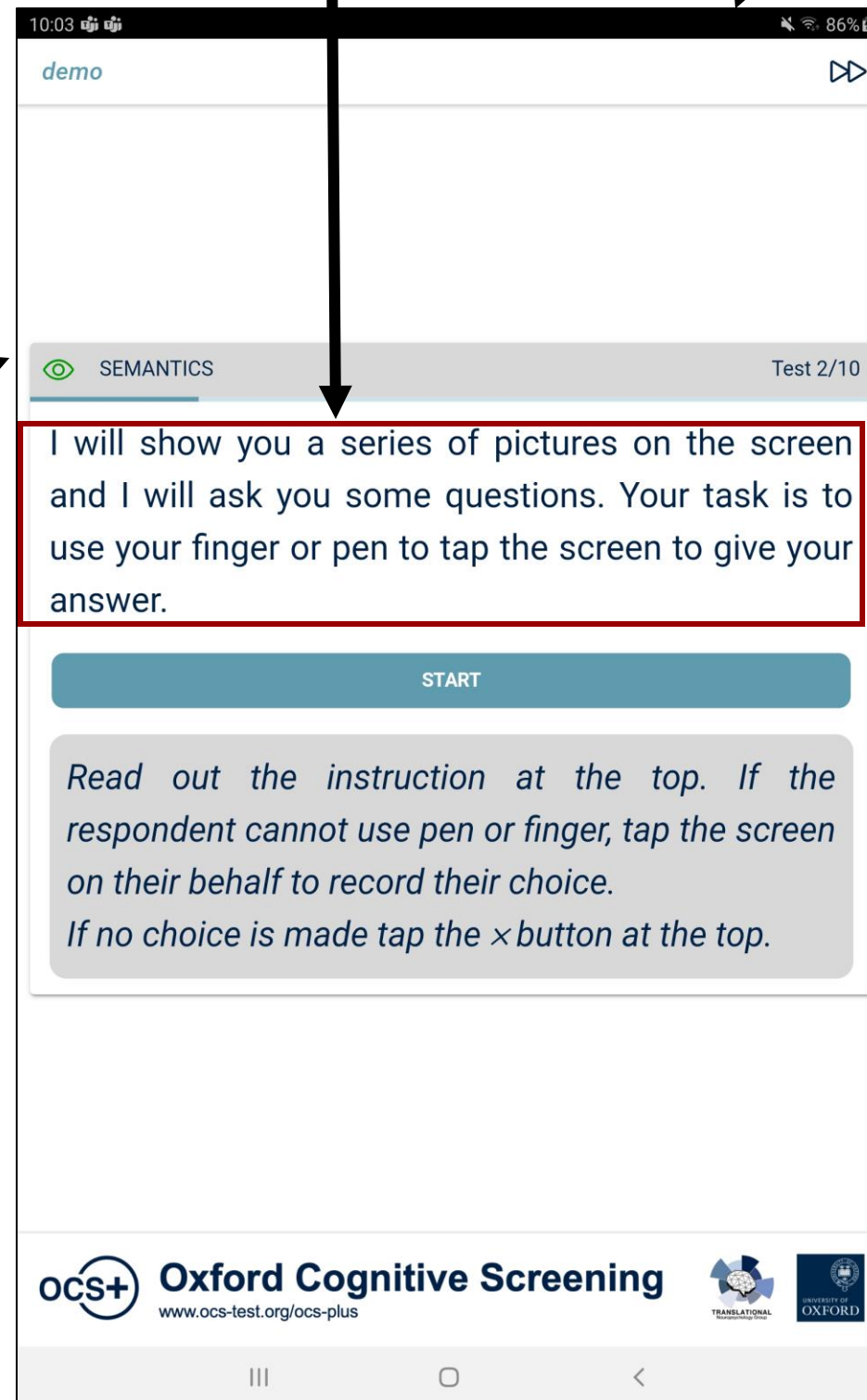


Instructions for **you**
- do **not** read aloud -

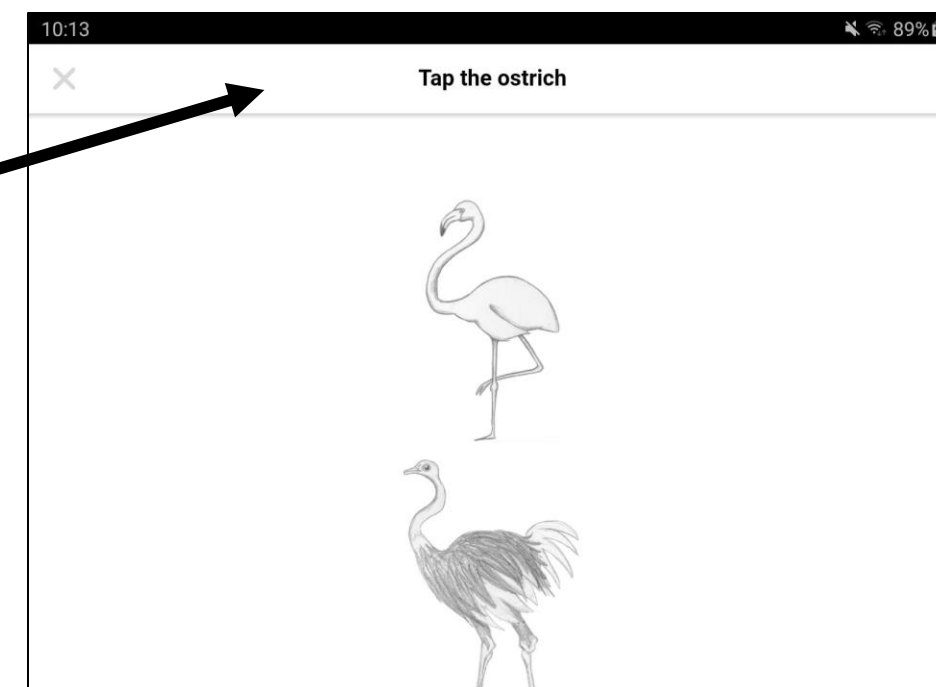


During subtests, **you** read aloud any text at top of screen



Completing an assessment

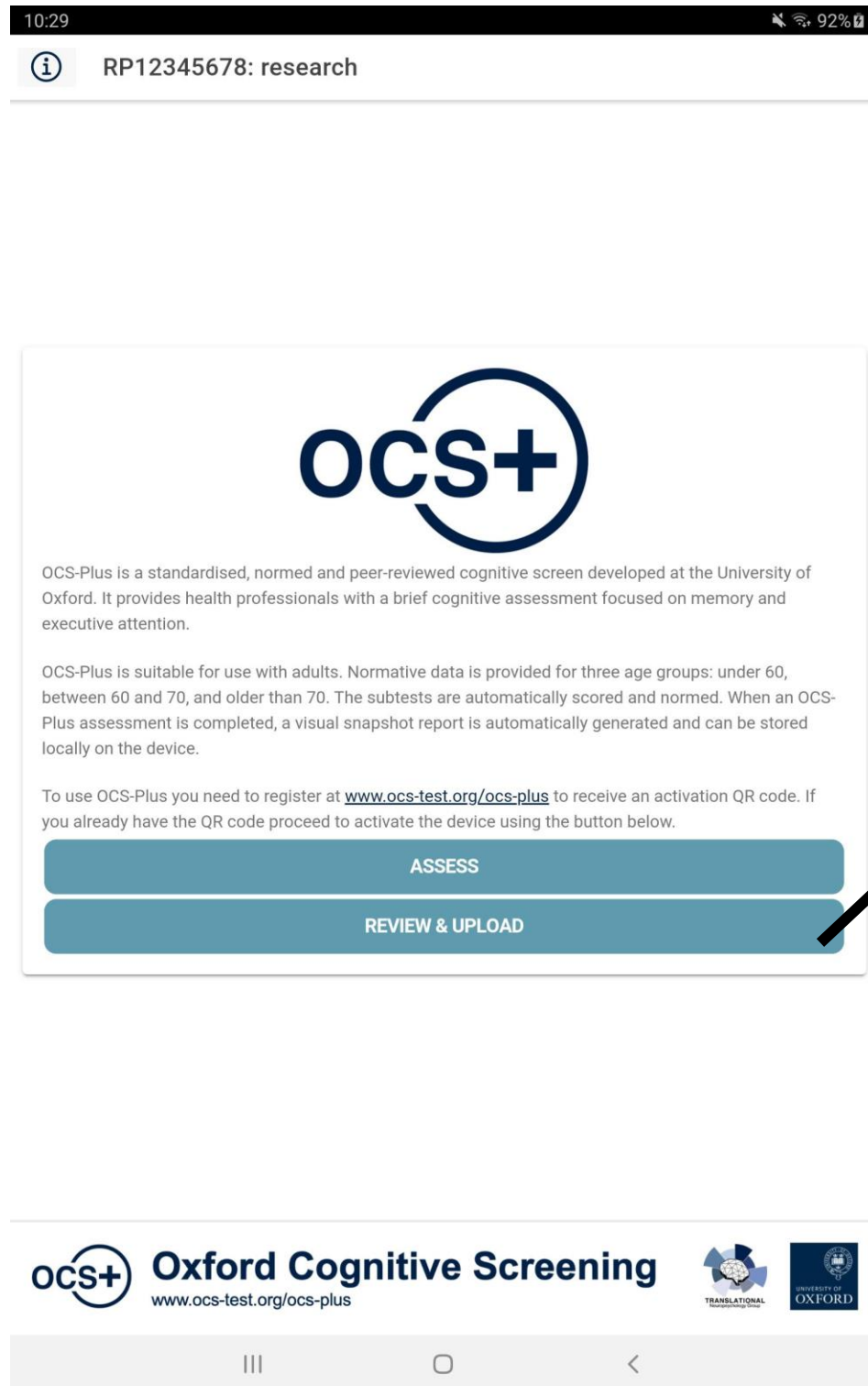When assessment is complete, you return to the home page to **review** complete and incomplete assessments




-  to view snapshot cognitive profile
-  to delete an assessment
-  to view assessment information, incl
Respondent ID
Start time/date
For incomplete: Resume from TASK
For complete: completion time/date

REPORT for an assessment

Respondent ID and age ←



Profile highlighting spared and impaired task performance

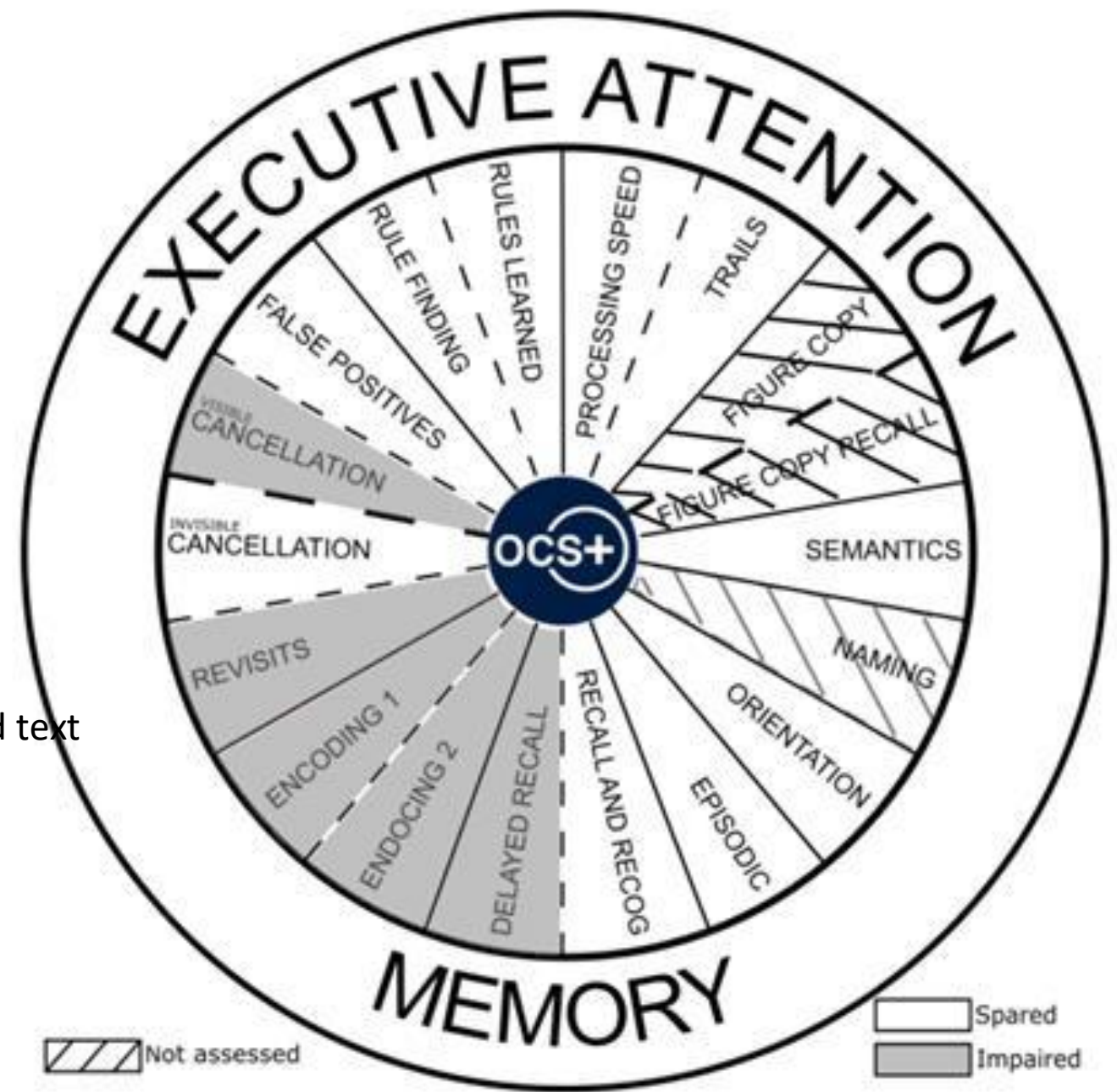

Click to add text

Note:

Currently figure copy not implemented with automatic scoring resulting in 'not assessed'

Licence number of OCS-Plus registration on each report ←

This report was generated under licence RP12345678
Please see www.ocs-test.org for further information.

Disclaimer: The information contained in this report is not in itself a clinical diagnosis. This report should be used to inform a clinical diagnosis that takes into account all relevant clinical factors.

 **Oxford Cognitive Screening**
www.ocs-test.org/ocs-plus

